BOS 2000-3

BRUKERVEILEDNING

BOS 2000-3 batteridrevet penispumpe
Vakuumterapi

Mai 2021

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Elektromagnetisk kompatibilitet (engelsk tekst)

EMC ACCOMPANYING DOCUMENT

The BOS-2000-3 requires special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in these accompanying documents.

WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the BOS-2000-3, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

WARNING: A risk of increased emissions or decreased immunity may result if any additional cables are attached.

WARNING: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

WARNING: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

1	Guidance and manufacturer's declaration – electromagnetic emissions		
2	The BOS-2000-3 is intended for use in the electromagnetic environment specified below. The customer or the user of the BOS-2000-3 should assure that it is used in such an environment.		
3	Emissions test	Compliance	Electromagnetic environment – guidance
4	RF emissions CISPR 11	Group 1	The BOS-2000-3 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are no likely to cause any interference in nearby electronic equipment.
5	RF emissions CISPR 11	Class B	The BOS-2000-3 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

Brukerveiledning

Guidance and manufacturer's declaration – electromagnetic immunity

The BOS-2000-3 is intended for use in the electromagnetic environment specified below. The customer or the user of the BOS-2000-3 should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8kV contact ±15kV air	±8kV contact ±15kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be a levels characteristic of a typical location in a typical commercial or hospital environment.

Recommended separation distances between portable and mobile RF communications equipment as well as RF wireless communications equipment the BOS-2000-3

The BOS-2000-3 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the BOS-2000-3 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BOS-2000-3 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)			
	80 to 800 MHz $d = [3.5/10] \sqrt{P}$	800 MHz to 2.7 GHz $d = [7/10] \sqrt{P}$	710, 745, 780, 5240, 5500, 5785 $d = [6/9] \sqrt{P}$	385, 450, 810, 870, 930, 1720, 1845, 1970, 2450 $d = [6/28] \sqrt{P}$
0.01	0.035	0.070	0.067	0.021
0.1	0.110	0.221	0.211	0.070
1	0.350	0.700	0.667	0.214
10	1.107	2.213	2.108	0.700
100	3.500	7.000	6.670	2.143

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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